

DiaDent[®]

DiaDent Group International
www.diadent.com | 1.877.342.3368

DiaPaste[™]

Premixed Calcium Hydroxide Paste
with Barium Sulfate Root Canal Treatment



ADVANTAGES:

- Premixed: save time!
- Stable: does not solidify nor separate
- Flexible pre-bent tips: easy access to apex
- Highly radiopaque
- Antibacterial
- Water soluble: easy to remove
- Biocompatible

INDICATIONS:

- Apexification
- Treatment for infected root canal
- Temporary root filling & pulp capping
- Root canal filling for primary teeth
- Disinfection after pulp extirpation
- Vital pulpotomy

The Brand You Trust
The Name You Know

INSTRUCTIONS FOR USE

• Root Canal Preparations •

Before using DiaPaste, clean and shape the canal using conventional methods.

***Note*:** the outside diameter of the tip is approximately equivalent to a #35 file or a 28 gauge tip.

• Temporary Intracanal Medicament •

Apply gentle pressure on the syringe plunger, the paste will begin to reach the apical foramen. At this point slowly withdraw the syringe. Once the canal is filled with DiaPaste, complete the treatment by placing a temporary restoration to seal the access opening. DiaPaste is water soluble and is easy to clean and remove.

• Deciduous Teeth •

Insert the syringe tip near the apex. Gently extrude DiaPaste into the canal from apex to coronal access. Take cautions to ensure excess DiaPaste is not extruded past the apex.



ORDER INFORMATION:

Regular Kit	(#A1001-401):	2.0 g of paste + 20 disposable tips
Refill Kit	(#A1001-404):	2.0 g of paste
Intro Kit	(#A1001-405):	0.5 g of paste + 4 disposable tips
Tips	(#1001-320):	20 clear pre-bent disposable tips