

DiaDent Absorbent Paper Points

Purpose of use

An absorbent paper points is an endodontic paper point that is used during the endodontic therapy to dry, or remove any foreign materials from the root canal of a tooth.

Directions for use

There are two stages to remove the moisture, blood and other foreign substance from the root canal. A dental professional has to follow steps to fill the root canal.

1. Before the stage of root canal filling with the Gutta Percha Points, the absorbent paper points should be checked the length and size to match the shaped root canal. An absorbent paper points should be selected to be fit to apex of root canal.
2. The selected paper points is inserted into the root canal to remove the blood or foreign substance before root canal filling permanently with Gutta Percha Points

This procedure is the most fundamental method. However, a dental professional may consider using more Absorbent Paper points to perform the procedure in a different way based on the patient's symptoms.

- ◆ Absorbent Paper Points are for a single use. Do not re-use them.


Handling & Storage

1. Store at room temperature. (2~27°C)
2. Do not throw the products while carrying
3. Store with separate from other products of different sizes and types

Caution

1. Sterile Paper Points should be used with the same size and thickness of the Gutta percha points.
2. Avoid contact with moisture.
3. Disposable medical products.

Symbols

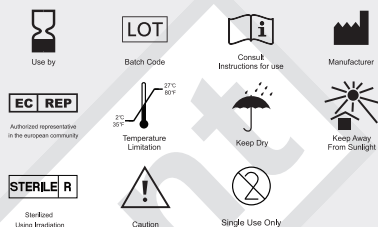

Shelf life

4 years from date of manufacture

Warranty and Limitation of Liability

Warranty

DiaDent Group International will replace product that is proven to be defective. DiaDent Group International does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

EC REP European Representative

DiaDent Europe B.V.
Antennestraat 70, 1322AS
Almere, the Netherlands
Tel: 00 31 (0)36 549 8607 FAX: 00 31 (0)36 536 7317

DiaDent Group International
16, Changjaengmyeong 4-ro, Osong-eup,
Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, 28161, Korea
Tel : 82-43-2862-2315
Fax : 82-43-282-8658
http://www.diadent.co.kr
E-mail : diadent@diadent.co.kr

DiaDent Group International Inc.
11-3871 North Fraser Way, Burnaby
B.C., Canada, V5J 5G8
Tel : 1-604-451-8851
Fax : 1-604-451-4865
http://www.diadent.com
E-mail : diadent@diadent.com

DiaDent Europe B.V.
Antennestraat 70, 1322AS
Almere, the Netherlands
Tel : 31-36-549-8607
Fax : 31-36-536-7317
http://www.diadent europe.com
E-mail : diadent@diadent europe.com

